



ADVANTAGE

TENNIS WHEELCHAIR



Engineered for **performance.**

THE STORY BEHIND ADVANTAGE

Designed and manufactured in South Wales, the Advantage Tennis Wheelchair represents the next evolution in athlete-focused wheelchair design.

Developed through collaboration with the Dan Maskell Trust, the Advantage has been engineered to give players greater adjustability, control and confidence on court.

Every detail has been designed with one goal in mind: helping athletes perform at their best.



Dan Maskell
Tennis Trust

Serving tennis to
disabled people

ENGINEERED FEATURES


The Advantage has been designed around the athlete, combining precision adjustment with reliable, competition-ready performance.

Performance Features

	26" Complete Leopard Wheels/Tyres/Tube/Pushrims
	72mm Castor Wheels
	½" QR Axle Pin
	Cross over Straps x2 (come as standard with the bucket seat)

Setup Tools Required


	Seat Removal 5mm Allen Key / 13mm Socket or Spanner
	Seat Height 6mm Allen Key / 13mm Socket or Spanner
	Footrest M4 Allen Key / 10mm Spanner
	Rear Castor Wheel Height Adjustment 19mm Socket
	72mm Castor Wheel Replacement 13mm Socket and Spanner

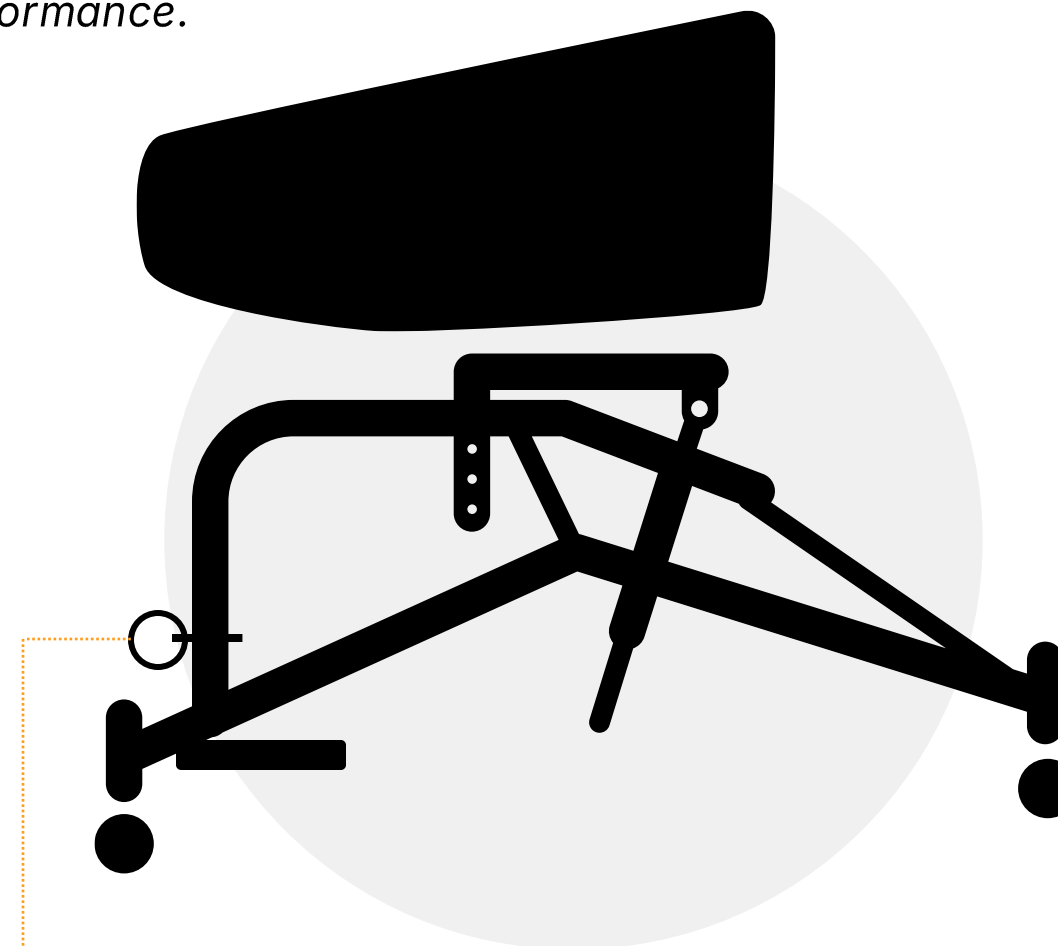

Quick Release Pin for Footrest Height Adjustment

PERFORMANCE GEOMETRY

Key frame measurements designed to support balance, control and court performance.

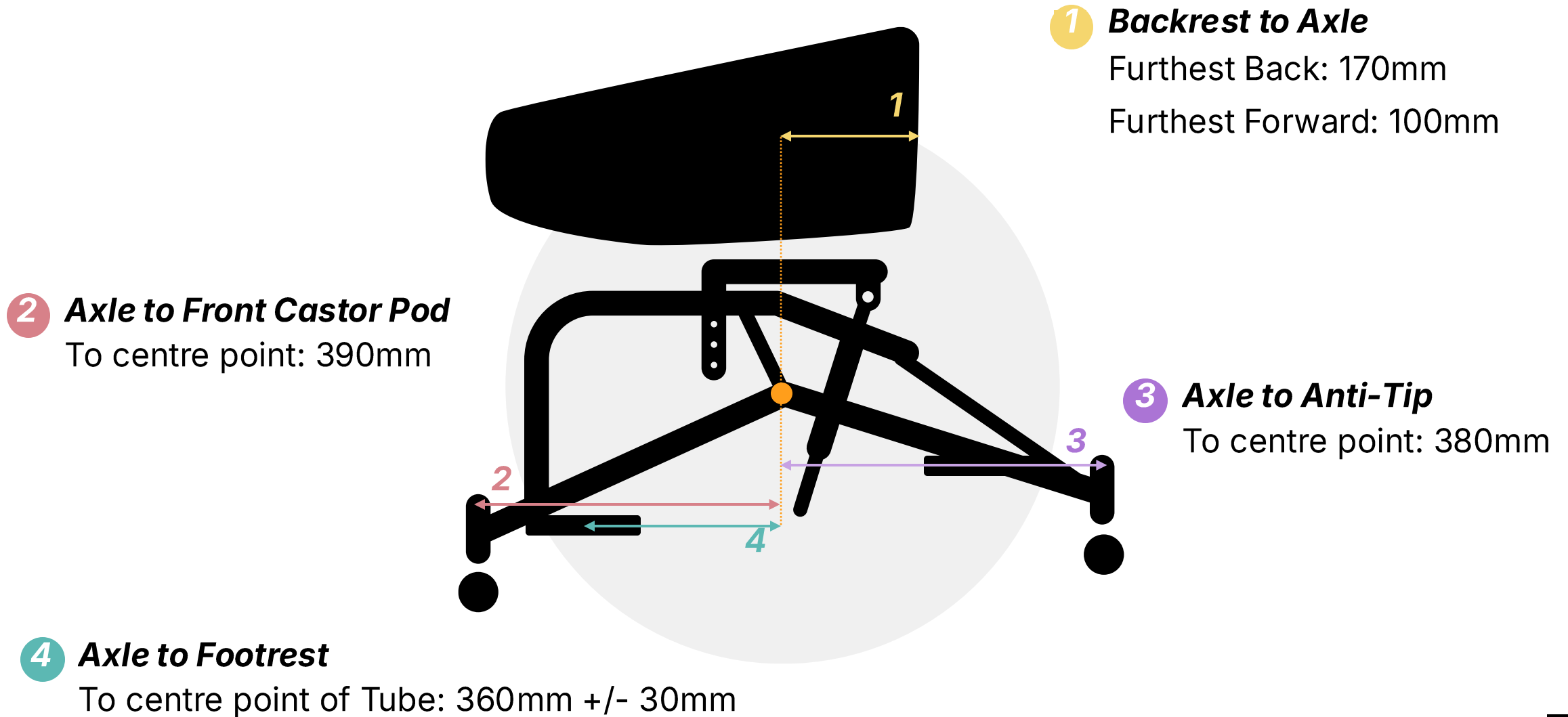


PERFORMANCE FIT

1 *Floor to top of footrest*
Lowest: 170mm

2 *Floor to top of footrest*
Highest: 120mm

3 *Floor to seat height*
Lowest: 460mm
(from front edge of seat)

4 *Floor to seat height*
Highest: 560mm
(from front edge of seat)

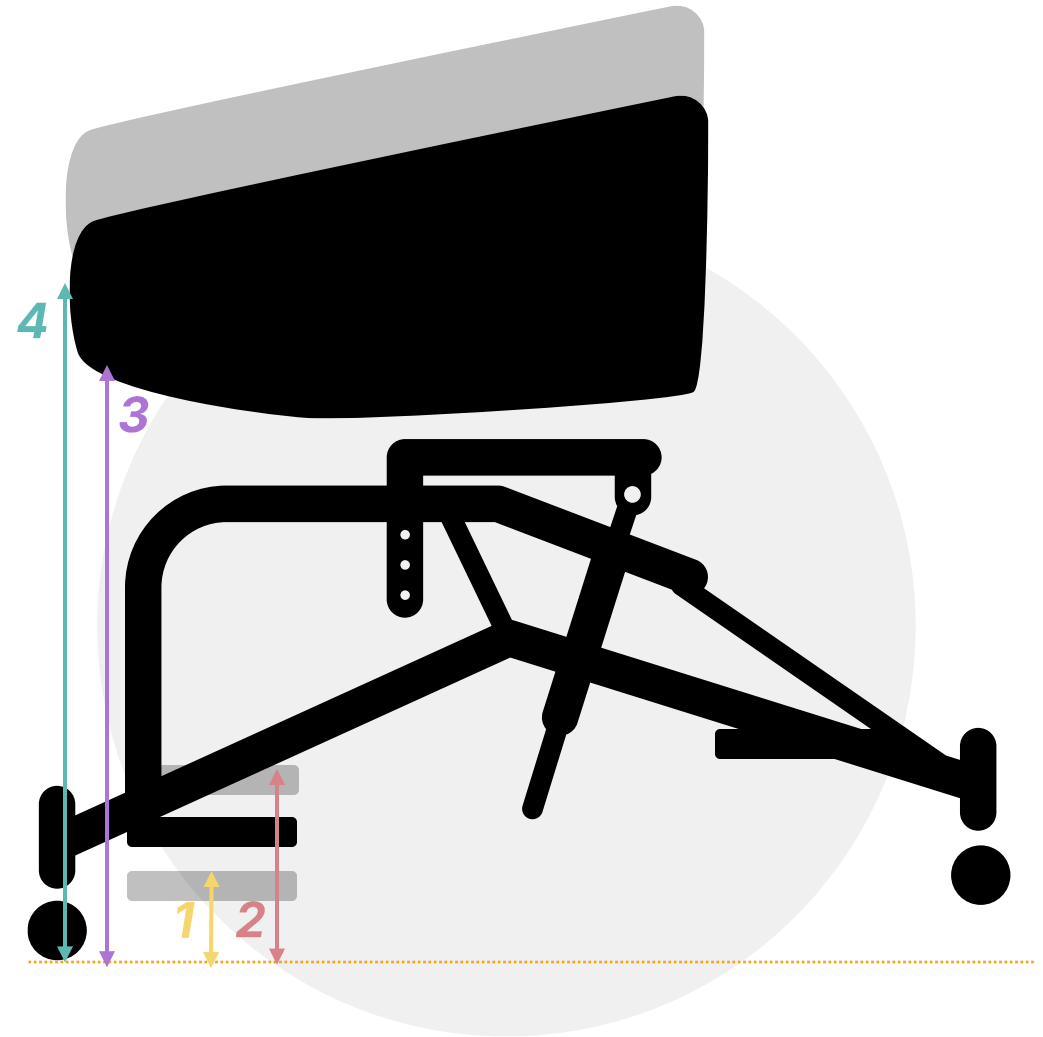

SEAT POSITIONING & CONTROL

1 **Seat Rake Adjustment**

Back: 17°

2 **Forward Seat Angle**

Forward: 20°


3 **Backrest Height Adjustment**

Based on this model: 220mm

Optional Extra: Custom Fitting high back (Chargeable)

4 **Footrest Angle Adjustment**

Unlimited pivot forward/back
(Fully Customisable)

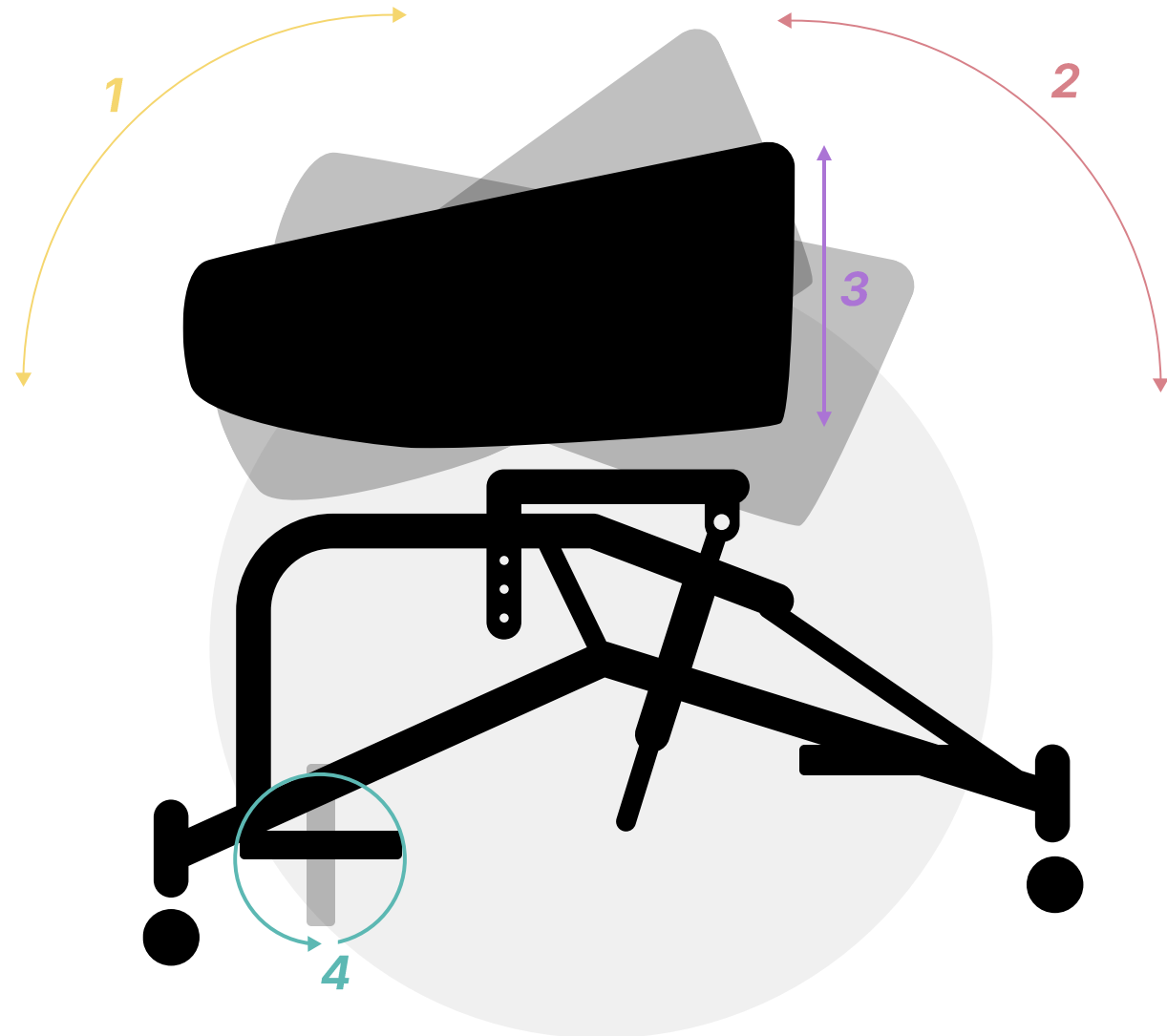

PLAYER CLEARANCE & SETUP

Designed to provide the clearance, stability and adjustability needed for individual playing styles.

1 **Width between front castors**

Centre to centre: 345mm

2 **Front seat down tube**

Clearance for legs: 230mm

3 **Seat width**

Available Widths: 12", 14.5" & 17"

Note: Measurement taken at base of seat.


4 **Wheel Clearance between Seat and Wheel**

Based on this model: 50mm either side

5 **Pushrim gap**

20mm / 10mm

Note: Please state at time of ordering

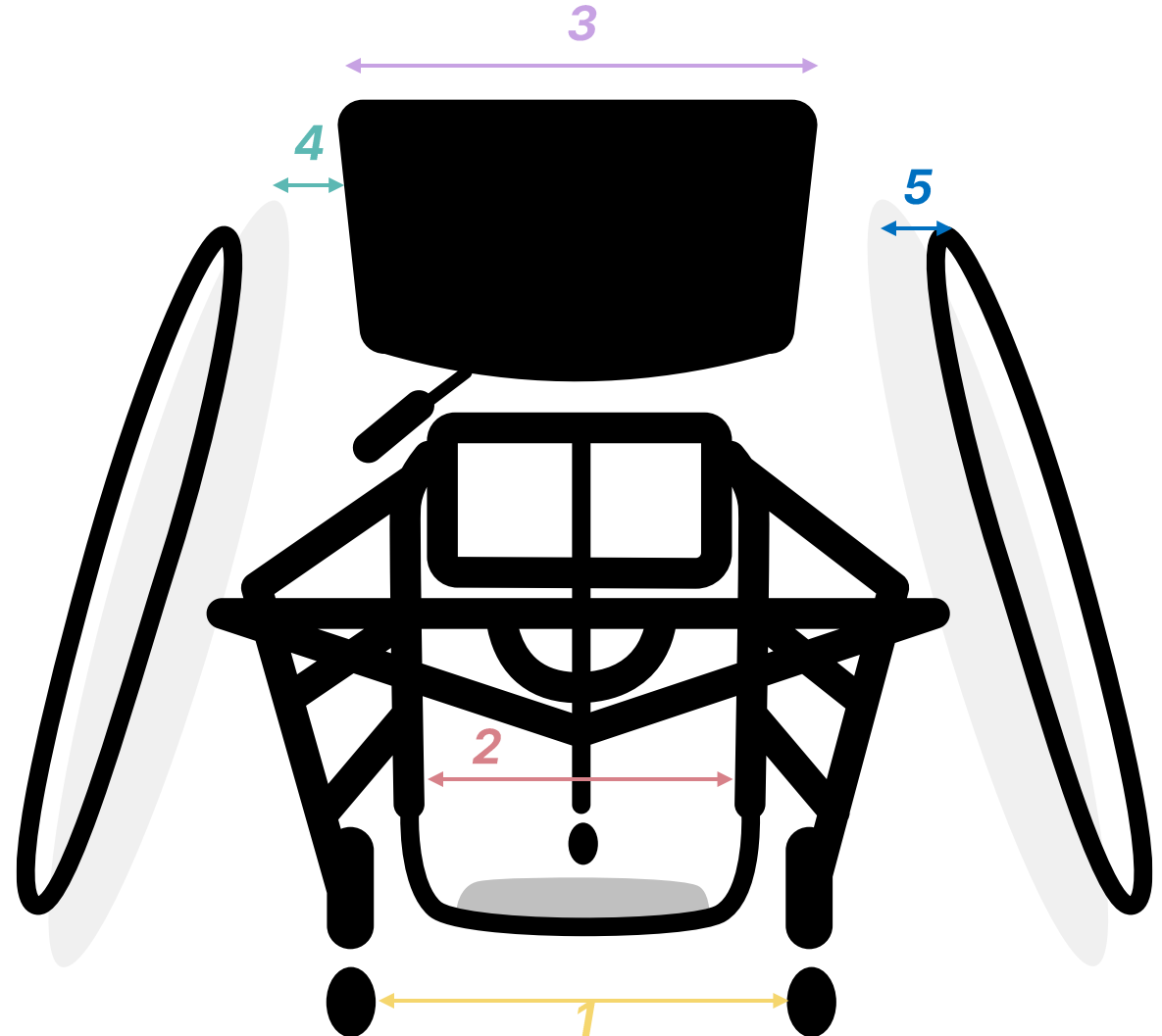

SEAT ANGLE SETUP




Fine tune the seat angle to achieve the preferred balance, stability and playing position.

- Turn clockwise to raise the seat angle.
- Turn anticlockwise to lower the seat angle.



SEAT HEIGHT & WHEEL SETUP

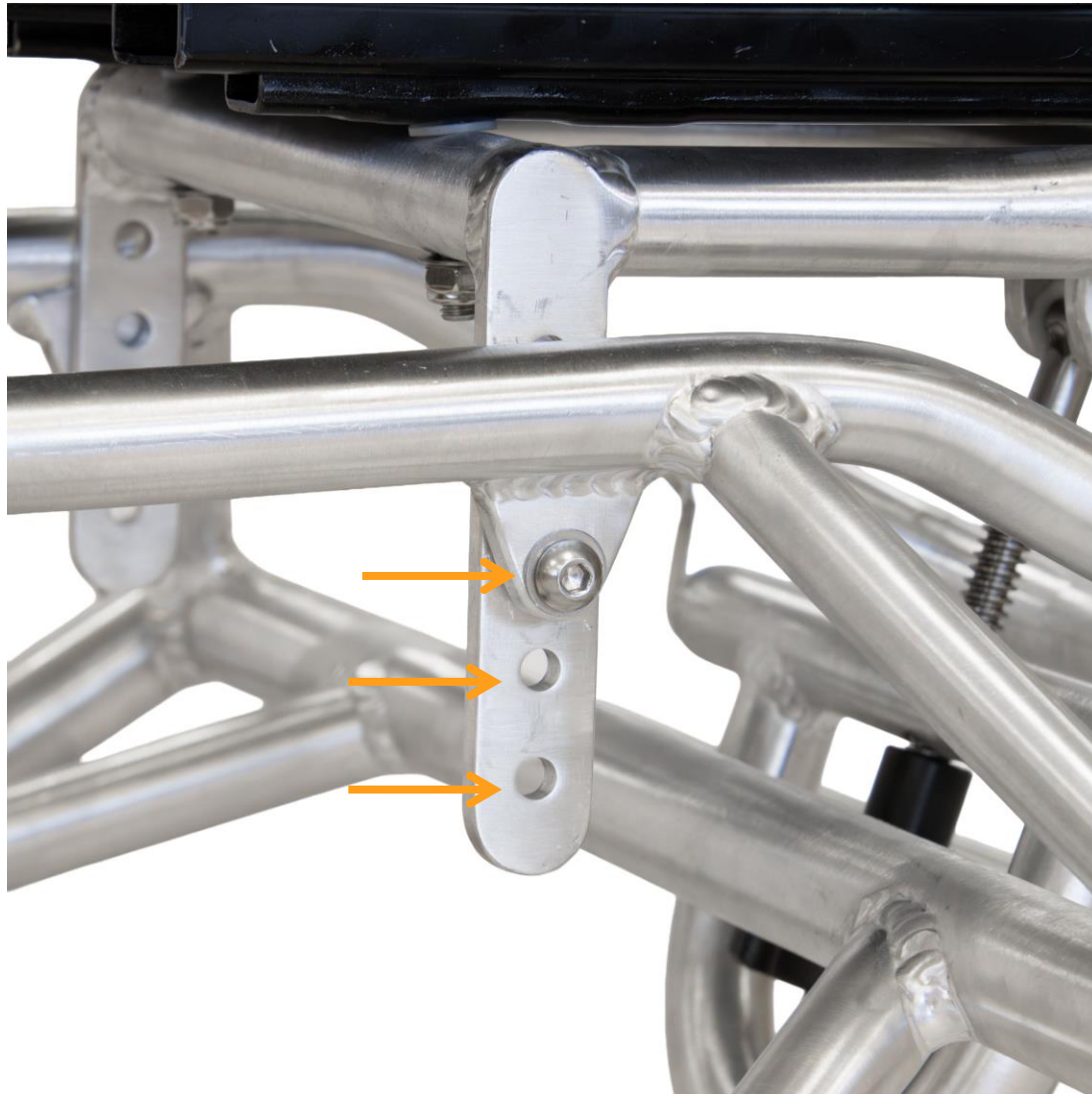

Three seat height positions allow the chair to be configured in 20mm increments.

26" Complete Leopard Wheels are fitted as standard for responsive, performance-focused movement.

SEAT DEPTH SETUP




Adjust the seat depth to create the ideal balance between agility, stability and power transfer.

Lift the handle and slide the seat forwards or backwards to suit the player's preferred setup for optimum position.



FOOTREST SETUP

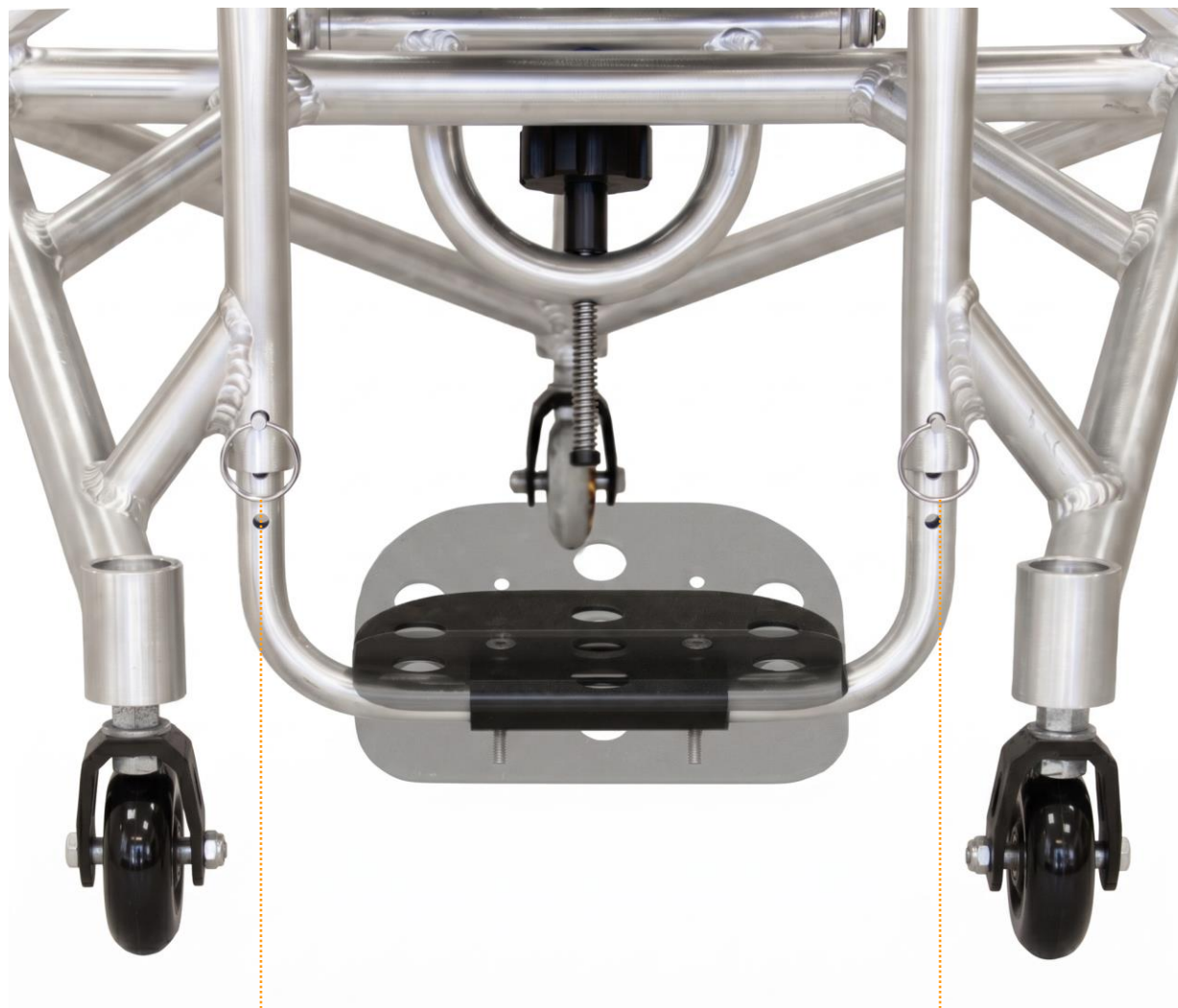

Quick Release Pin for Footrest Height Adjustment

Position the footplate to support comfort, stability and efficient propulsion.

Adjust the footplate angle using an Allen key.



PERFORMANCE GALLERY

FRONT VIEW

Engineered for performance from every angle.



PERFORMANCE GALLERY

REAR VIEW

Built for stability, control and confidence on court.



SPARE & REPLACEMENT PARTS

The parts listed below are approved spare and replacement components for this wheelchair. Please quote the relevant part number when ordering. Please note that all parts are chargeable.

Part Numbers	Description
26006	26" Leopard Roder Wheels C/W T26806 and TB261006
WH0049	72mm Castor Wheel
AXL ½ STEEL	½" QR Axle Pin
PL2903	12" Plastic Bucket Seat
PL2904	14.5" Plastic Bucket Seat
PL2911	17" Plastic Bucket Seat
UP0074/19	19" Foot Strap
ASSY004	Front Castor Wheel Setup Complete / without 72mm Wheel
ASSY003	Rear Castor Wheel Setup Complete / without 72mm Wheel
SSQRP0630	S/Steel Quick Release Pin for Footrest Height Adjustment
204901-88803-A2	Seat Slider Left Hand
204902-88803-A2	Seat Slider Right Hand with Handle
TP0612	Footrest Pivot Bracket